**Produktdatenblatt**  
**Print@Dreve Face Shield**

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Herewith we declare in our own responsibility, that the product described below placed on the market by us is in conformity with the Regulation (EU) 2016/425 and fulfil the applicable essential health and safety requirements set out in Annex II. In case of any application of the product not authorised by us this declaration becomes invalid.

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  
Dr. Reiner Altmann  
Leiter Qualitätsmanagement & Regulatory Affairs  
Head of quality management & Regulatory Affairs

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
<b>Classification</b>	Category II (No.6) of Regulation (EU) 2016/425 for personal protective equipment (PPE)
<b>Product description</b>	Face protection for continuous use. Certified to EN166:2001 as protection against liquid splashes. Does not provide protection against mechanical impact.
<b>Material</b>	Headband: elastic rubber braid, width: 15mm Support: FotoDent <sup>®</sup> model2 Screen: PETG; thickness: 0.5 - 0.625 mm
<b>Area of application</b>	Face shield for medical personnel for protection against liquids and droplet transfer via air. The user can replace the screens with replacement screens. See instructions "Changing the screen".
<b>Storage</b>	Without direct sunlight, store dry in original packaging Shelf life of the FaceShield: 1 year Shelf life of the screen: 5 years
<b>Cleaning</b>	For cleaning only the Dreve - cleaning liquid (REF471) should be used. Apply the cleaning liquid spraying on the front of the screen and distribute evenly with a soft cloth. Repeat the procedure for the back of the screen.
<b>Disposal</b>	The product can be thermally recycled or can be disposed of in landfills, no toxic substances are released. In the case of contaminated products, the type and extent of the contamination determines the disposal, whereby the applicable laws and regulations must be observed.

# Product data sheet

## Print@Dreve Face Shield

### Setup – Scheme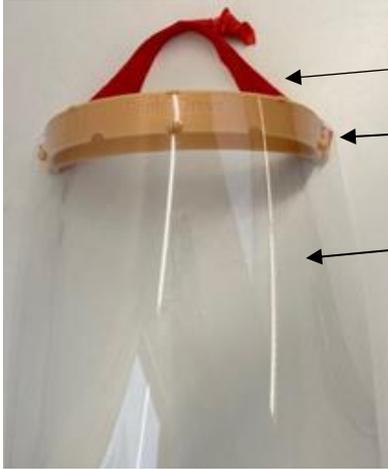


Headband for fixing and positioning the screen in front of the face

Support for holding and positioning the screen in front of the face


Screen to protect the face from liquids and splashes

### Montage

#### 1.) Attaching the headband

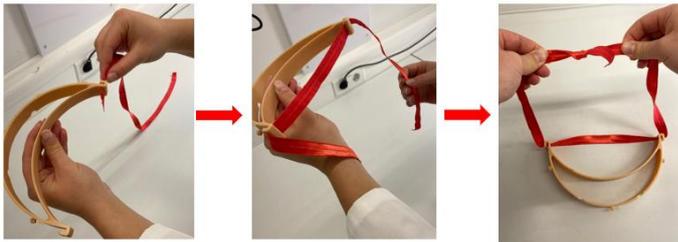#### 2.) Remove the protective film from the screen and assemble the screen on the support.
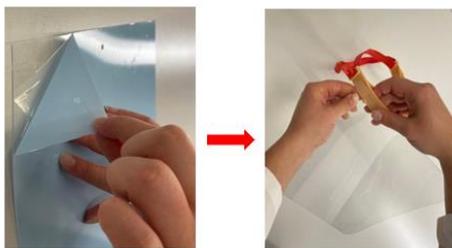
#### 3.) Put on face shield


# Product data sheet

## Print@Dreve Face Shield

### Changing the screen

- 1.) Remove the old screen with scissors in order not to damage the retaining pins on the support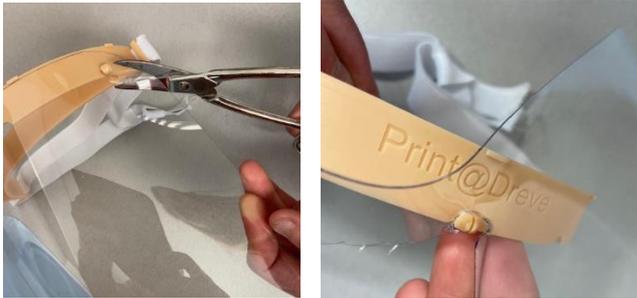

- 2.) Position the replacement disc (REF M190001). Start by fixing the central opening to the central retaining pin of the support.

- 3.) Remove the protective film on the front and back of the screen before first use

Hereby Innovation MediTech GmbH declares that the Print@Dreve Face Shield complies with regulation (EU) 2106/425. The complete text can be found at the following Internet address: <https://innovation-meditech.de/printdreve/>.

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*Herewith we declare in our own responsibility, that the product described below placed on the market by us is in conformity with the Regulation (EU) 2016/425 and fulfil the applicable essential health and safety requirements set out in Annex II. In case of any application of the product not authorised by us this declaration becomes invalid.*

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**Notifizierte Stelle (Modul B):** ECS GmbH, Identifikationsnummer (ID) 1883
**Notified body (Module B):** ECS GmbH, Identification number (ID) 1883
**Unterzeichnet für und im Namen von:** Innovation MediTech GmbH
**Stempel:** INNOVATION MEDITECH GmbH Unna, 2020-04-21